



Instruction for use : Surface to be treated should first be compressed with dry gauze, remove the gauze carefully, and immediately cover the surface with Hemocoll. The amount required will depend on the force applied and severity of the bleeding. It is necessary to apply moderate pressure over the Hemocoll with a dry gauze.

Mfg. Lic. Number : 675

Manufactured & Marketed by:



Advanced Biotech Products (P) Ltd.

77, First Cross Street,
Ragavan Colony, Chennai - 600083, India.

Under the Licence from :

ENCOLL
Fremont, CA, USA

Phone: +44-4352-1681 Email: info@abpdental.com
Cell: +91-9176999850 Website: www.abpdental.com

HEMOCOLL™

**THE ABSORBABLE STERILE FIBRILLER COLLAGEN WOUND FILLER
FOR
DENTAL SURGERY
HEMOCOLL IS NON TOXIC AND NON PYROGENIC**




Advanced Biotech Products (P) Ltd

www.abpdental.com

info@abpdental.com

HEMOCOLL™

Hemocoll is high purity type I reconstituted Collagen

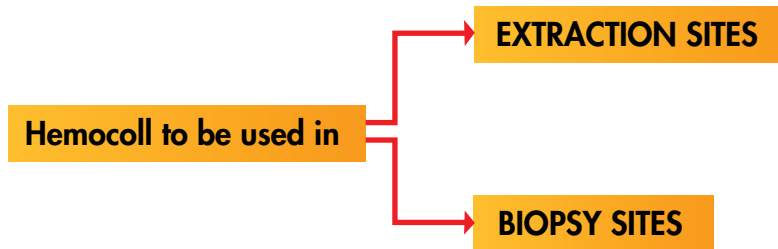

Benefits of Hemocoll:

1. Controls bleeding and stabilizes blood clots (with in 2 to 5 mins).
2. Protects wound bed.
3. Helps in regeneration of lost Tissue.
4. Resorbable in 3-4 weeks.
5. Faster healing process.

Hemocoll : Available in the form of dry fibrillar collagen
individual pack size of 0.25gm
(Approx. 2.5ml) & 0.50gm (Approx. 5ml)

Precautions:

- ❖ Hemocoll is not to be used on the infected or contaminated wounds.
- ❖ Not intended for use in treating systemic coagulation disorders.
- ❖ Do not store above 32° C (90°F).
- ❖ Discard all open and un used dressing.
- ❖ Do not attempt to resterilize.
- ❖ Safety in pregnancy not established.